

iNAP Lab+ APP Instructions

Date : 2020-05





First Time Use

Accept privacy policy to
allow device permissions.

Location allows
Bluetooth scan.

Storage permission
allows data recording.

Camera permission
allows QR code scanning
and take photo for
personal information.

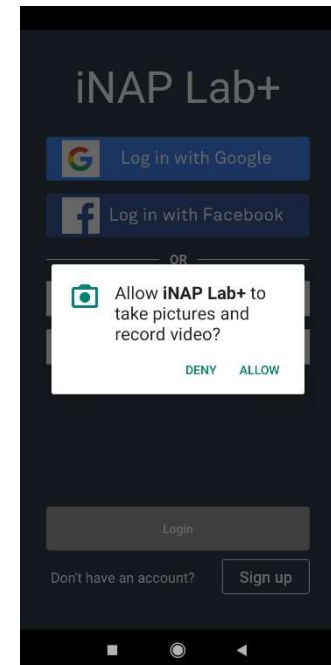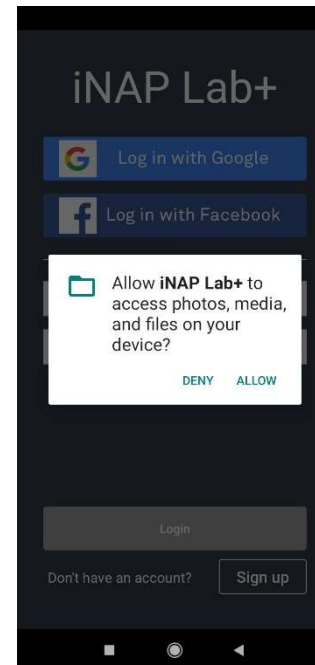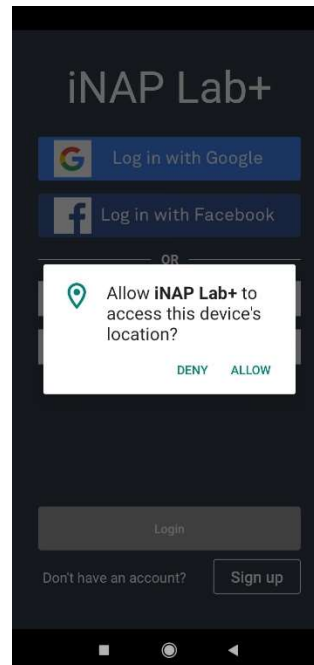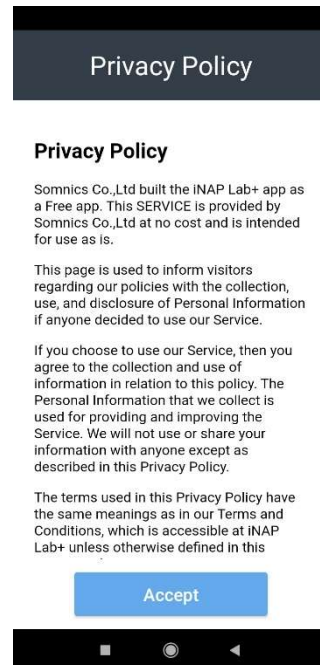

Sign Up

Click “Sign up” button to enter.

Key in verifiable email and password.




Confirm password and click “Sign up” button.

The image displays two mobile app screens for the iNAP Lab+ application. The left screen, titled "iNAP Lab+", features login options for Google and Facebook, a "Login" button, and a "Sign up" button. The right screen, titled "Sign up", prompts the user to "Create your iNAP account" and includes input fields for email (pre-filled with "somnia@somnia.com"), password, and confirm password, along with "Sign up" and "Cancel" buttons. An orange arrow indicates the transition from the "Sign up" button on the left screen to the sign-up form on the right screen.

Login

Login with Google or
Facebook account.

Or, login with signed
up email.

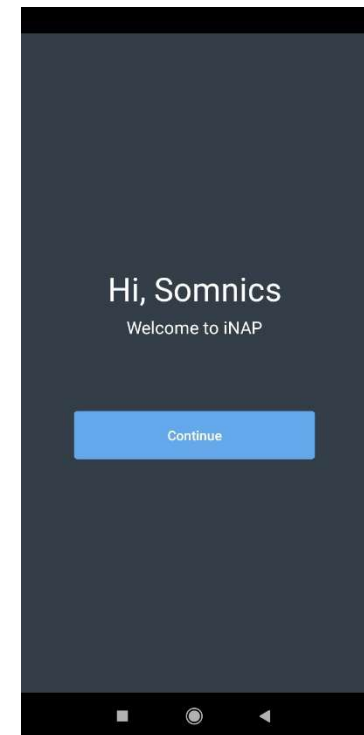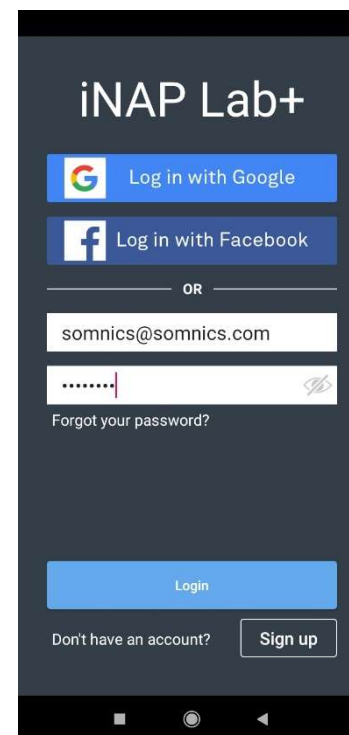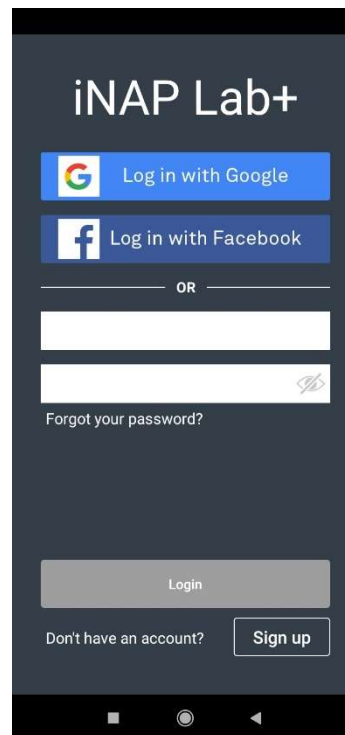

Forgot Password

If you forget your password, just click “Forgot your password?”

Key in your account email and click “Submit”.

An email will support you to find your password.

The image displays two mobile app screens side-by-side, illustrating the forgot password process.

Left Screen (Login Page):

- Header: iNAP Lab+
- Buttons: Log in with Google, Log in with Facebook
- Text: OR
- Input fields: Email and Password
- Link: Forgot your password? (An orange arrow points from this link to the right screen)
- Buttons: Login, Sign up
- Text: Don't have an account?

Right Screen (Forgot Password?):

- Header: Forgot Password?
- Text: Request a password reset for your account. An email will be sent to you with instructions.
- Input field: somnics@somnics.com
- Buttons: Submit, Cancel

Home Page

Home page display treatment time and the sealing time of the last treatment.

Results:

Treatment results.

Device list:


Check devices listing.

Compliance Download:

Download compliance records from the iNAP device. Upload records to the cloud server and erase all records in the iNAP device.


Factor:

You can select multi factors according to record pre-use daily routine.

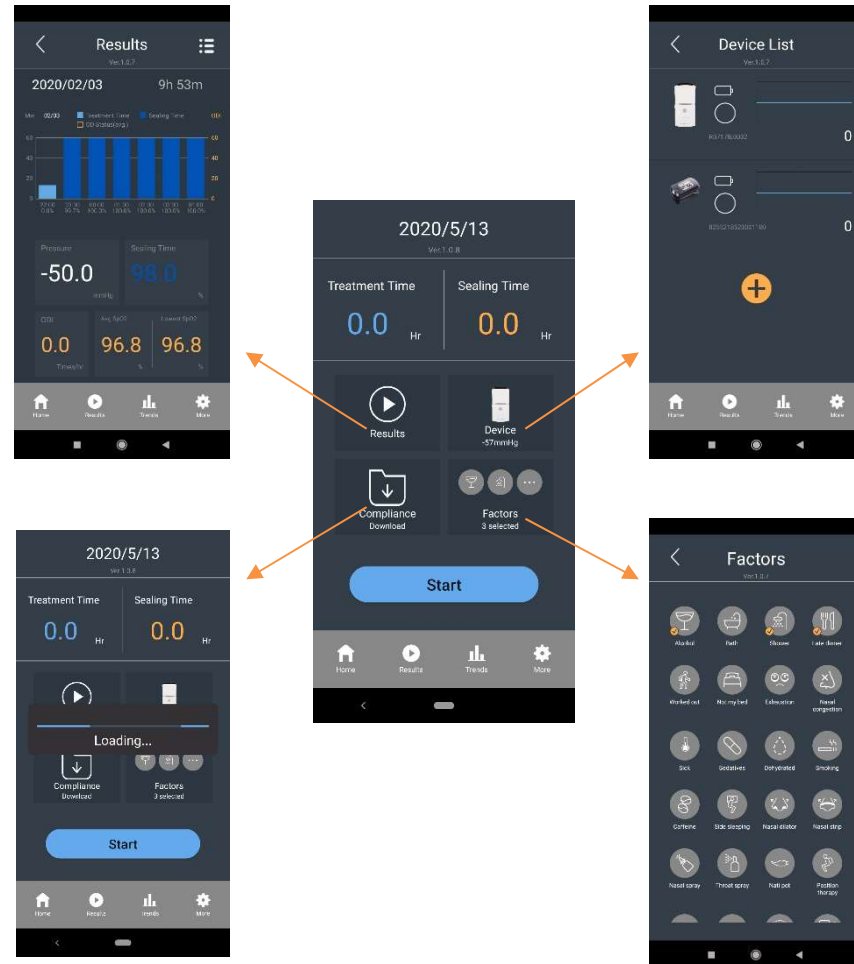

Pair Device – Part 1


Add new device by scan QR code or select the device serial number from list by Bluetooth scan.

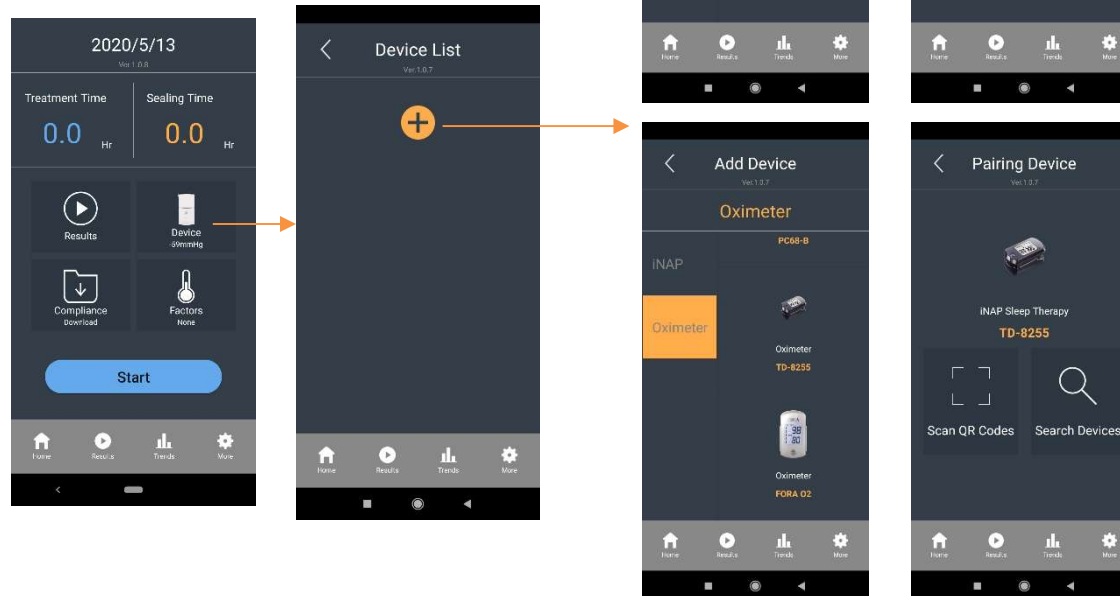

Pair Device – Part 2


The serial number of device will be displayed on the scan list.

Choose the device. If pairing is successful, click “Continue” to complete the pairing.

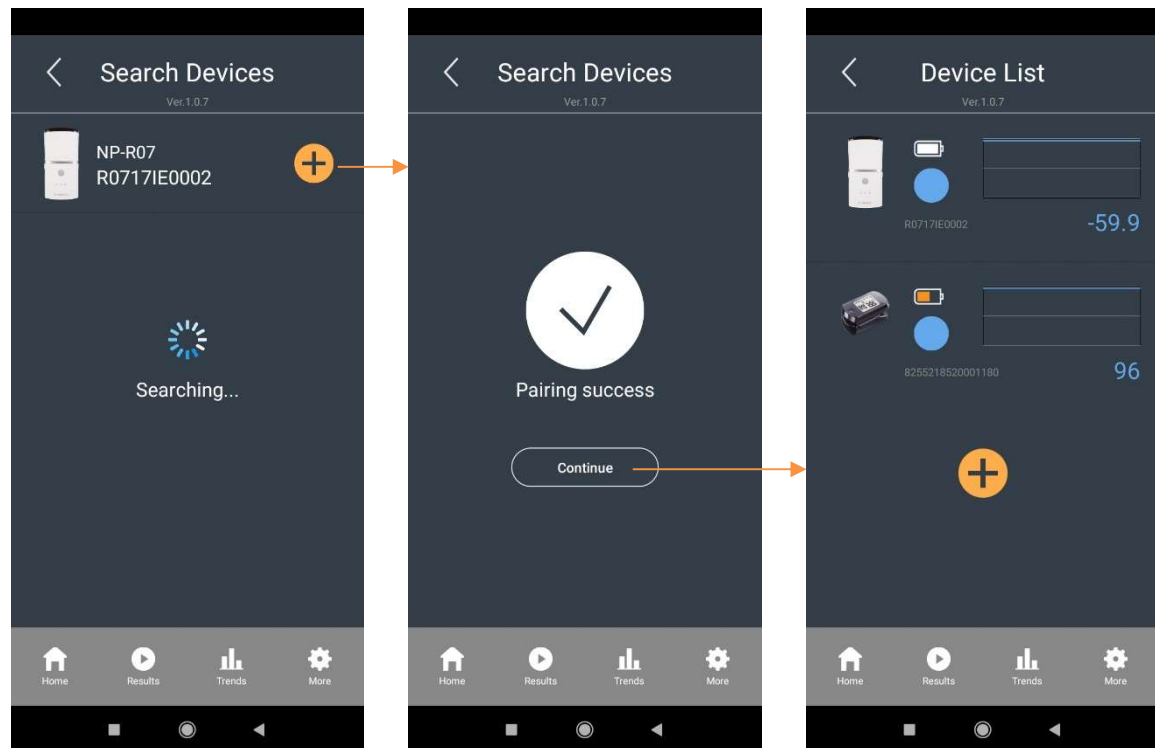


Pair Device – Part 3

If the pairing failed, please
retry or cancel.

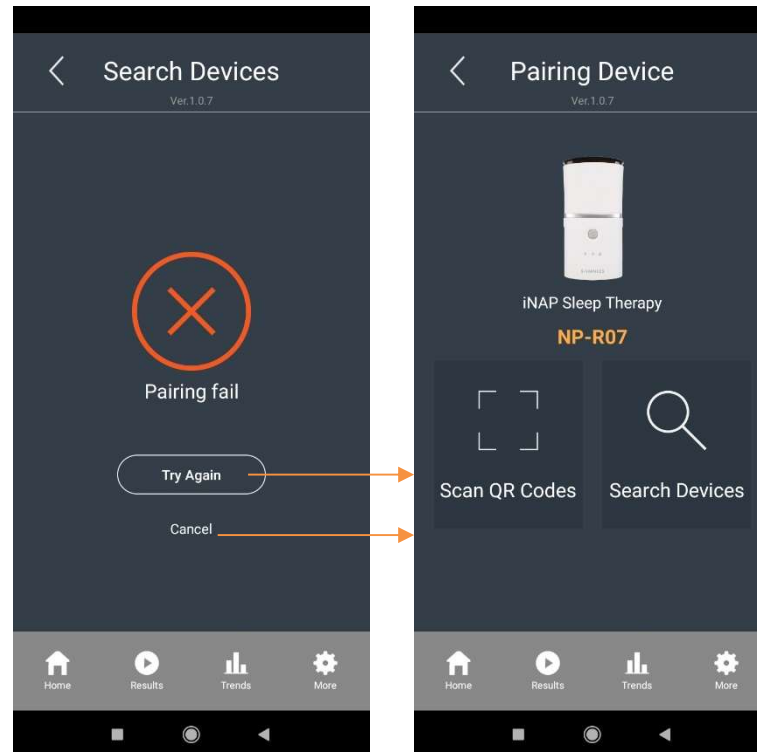

Device List

To delete a device from list,
slide that device to the left.
Click "Delete".

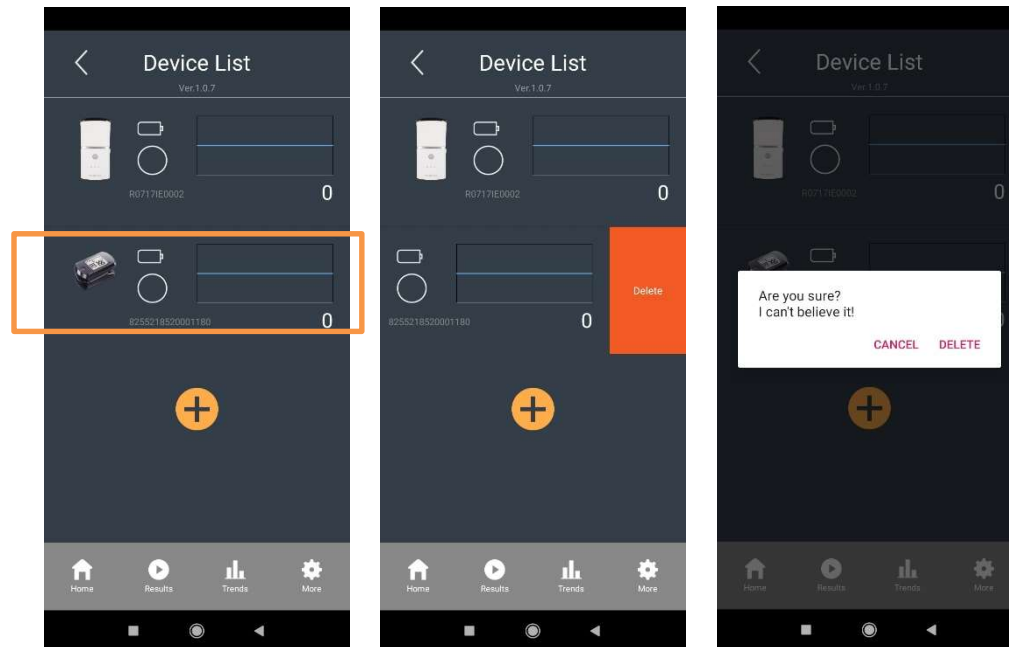

iNAP Device Support



R03



R07

Oximeter Device Support



FORA O2



PC68B

Start Record




Before start recording, make sure device is connected.

Click “Start”, and iNAP Lab+ will continuously record the data of the device under operation.

Do not turn off your Bluetooth and avoid to use other apps simultaneously.

Slide the arrow to stop recording, and record will be uploaded to cloud for AI computing.


It is recommended to connect the cell phone to power adaptor during recording.

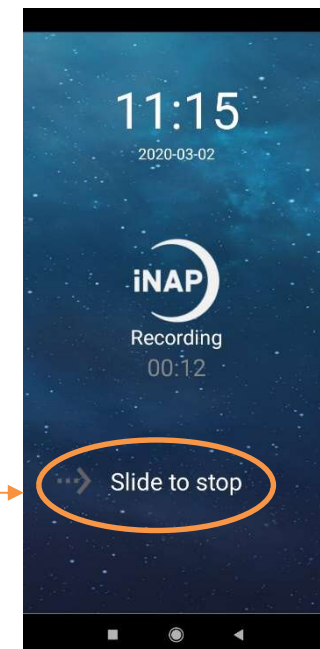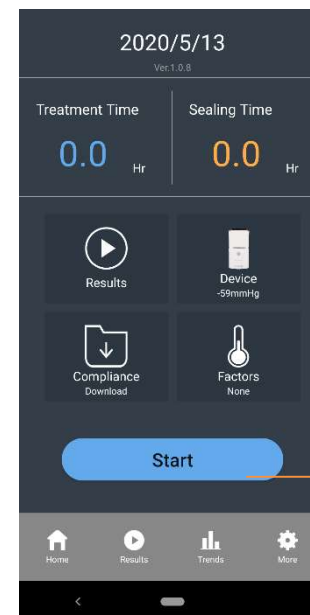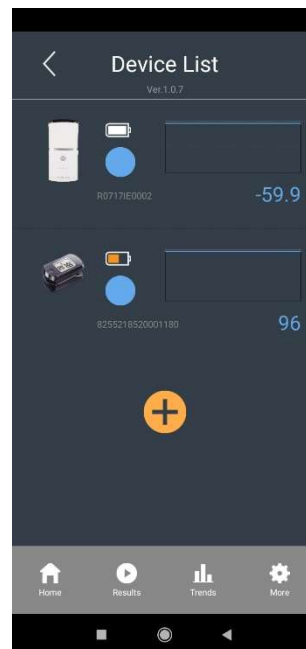

Record Results

The result page shows the data downloaded from the cloud.


Click on the upper righthand corner to check more historical records.

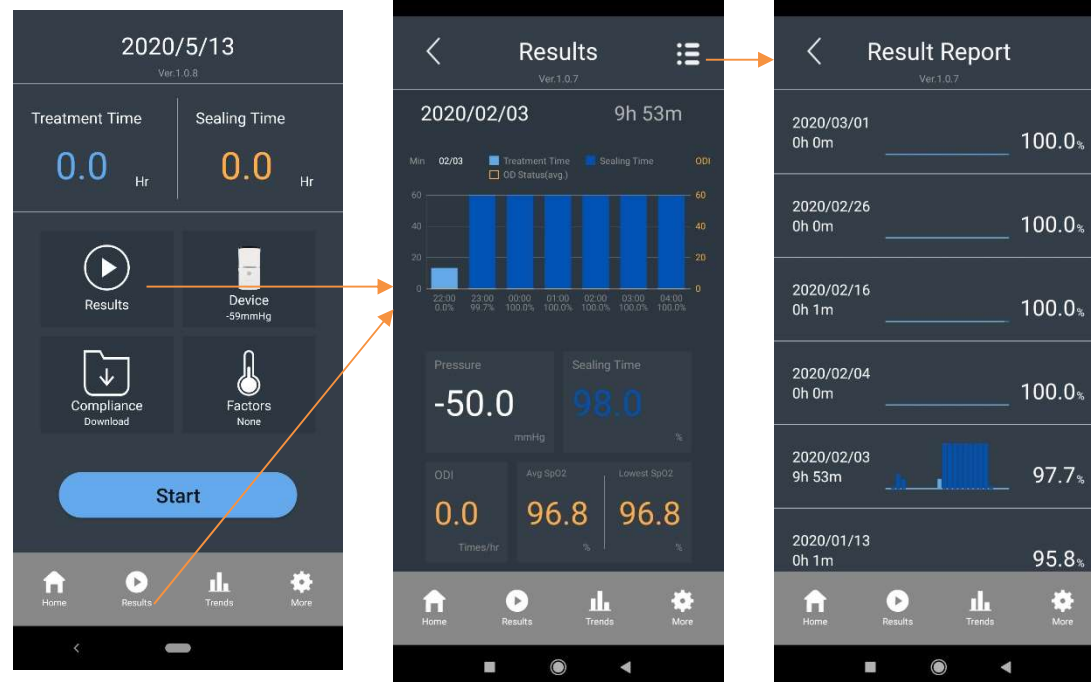

Record Trends



The trends page shows the data downloaded from the cloud.

You can select the records of compliance and ODI, and you can select by daily, weekly, or monthly record trends.

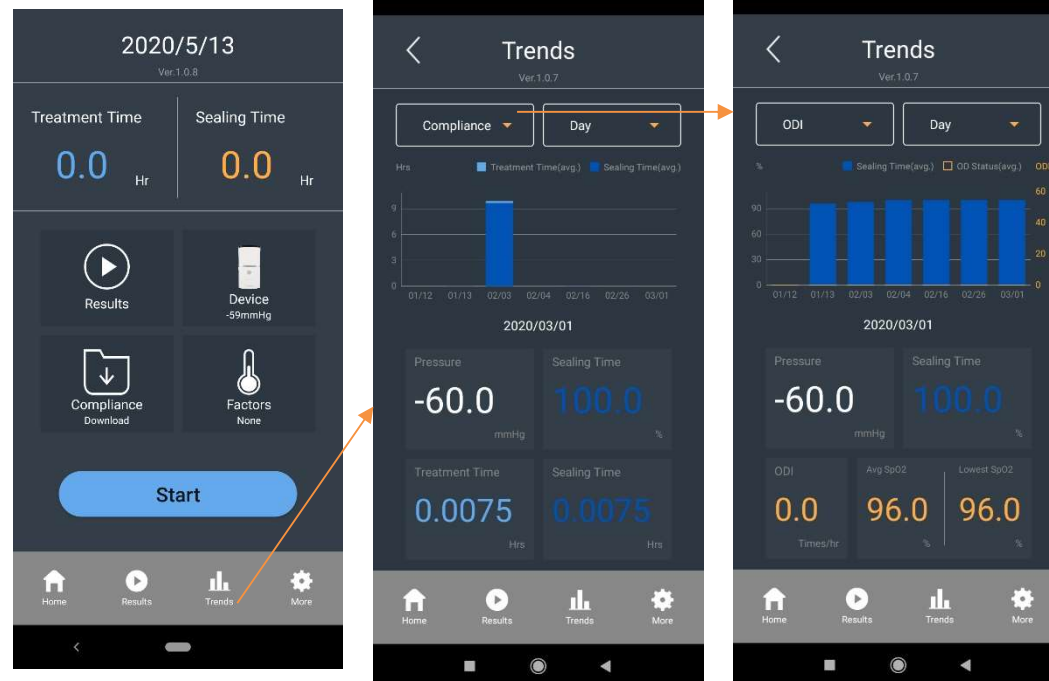


More – Personal Info

You can edit your account's personal information photo, height, and weight.

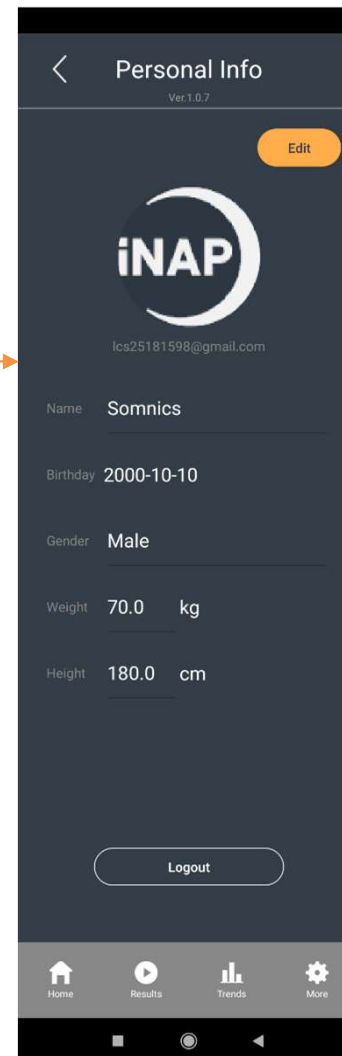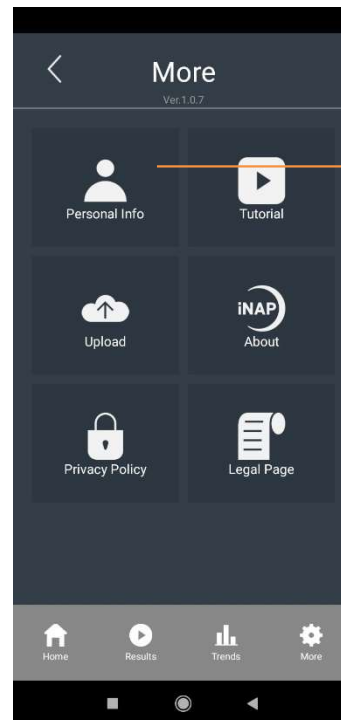


More - Tutorial

You can watch User's Guide and tutorial video of iNAP device.

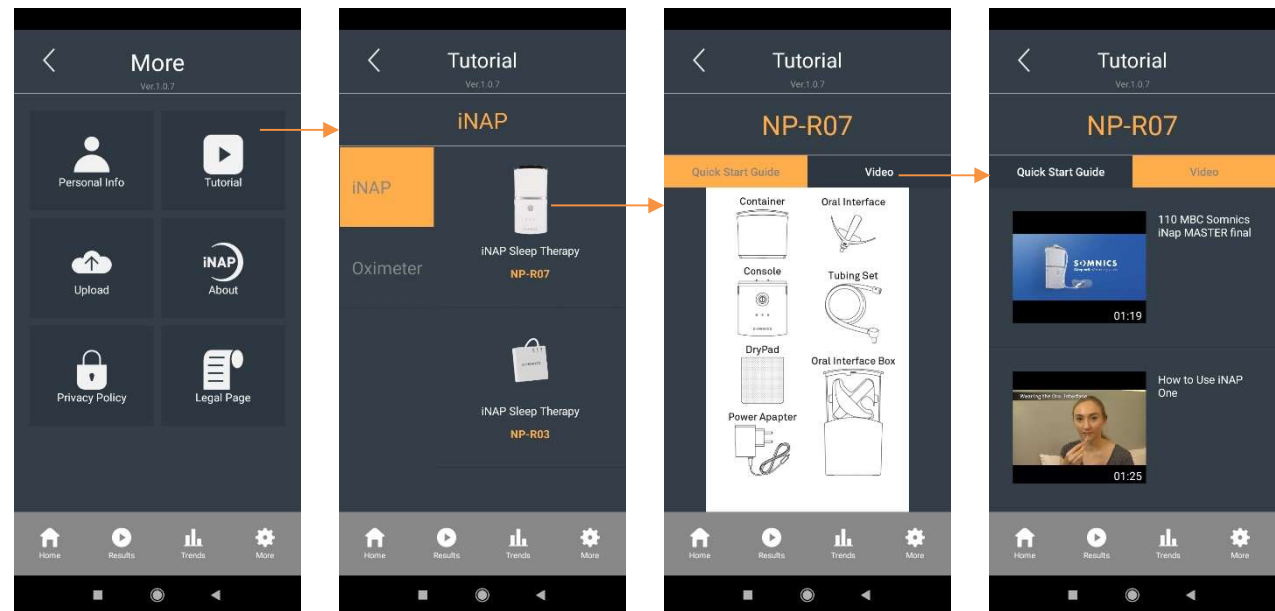

More - Upload



Each treatment record status will be displayed on upload page.

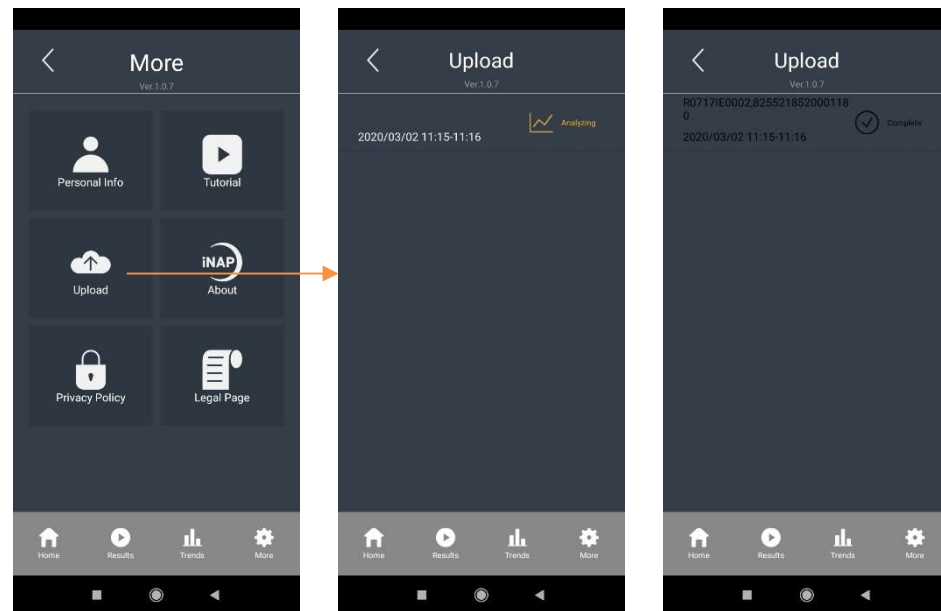



More - About

Contact Somnics @ about
page.



iNAP Lab+ version.

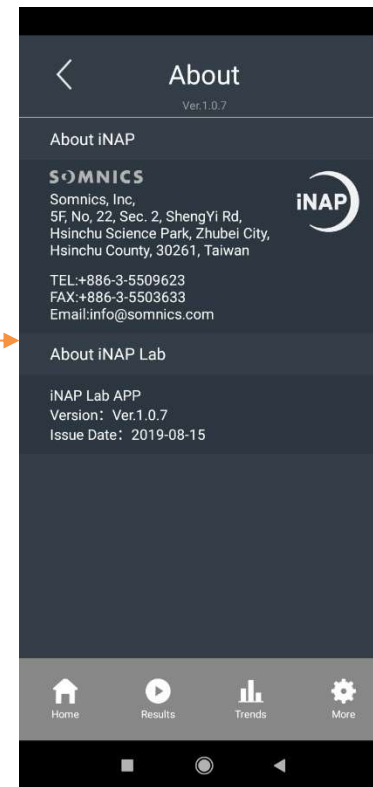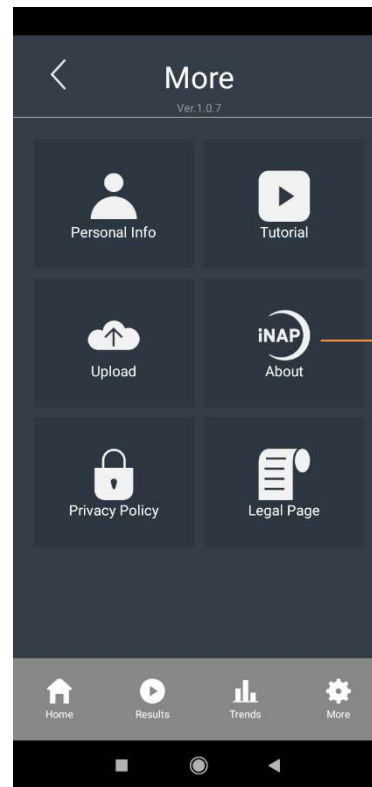


More – Privacy Policy

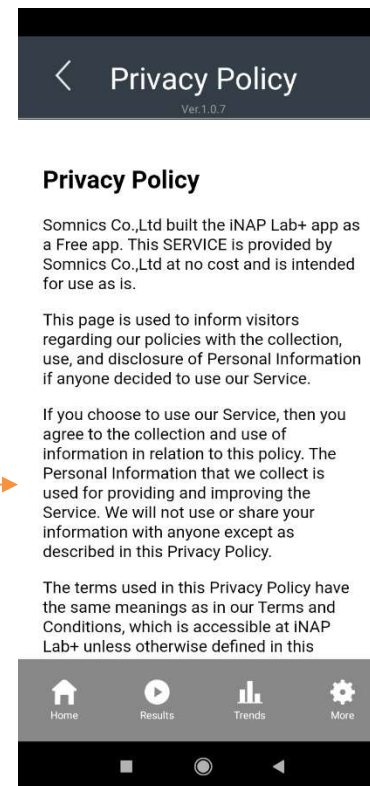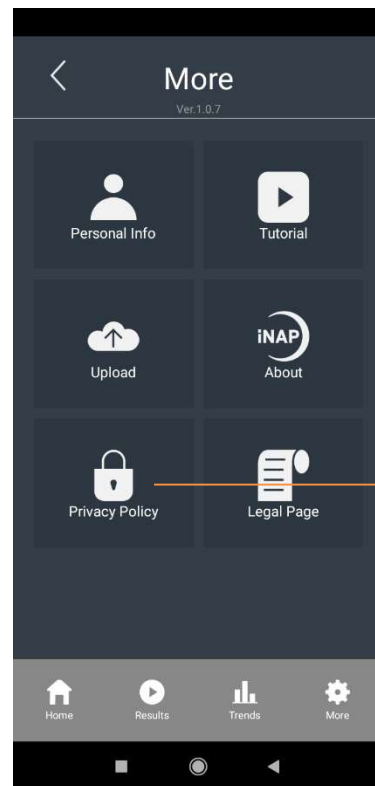

More – Legal Page

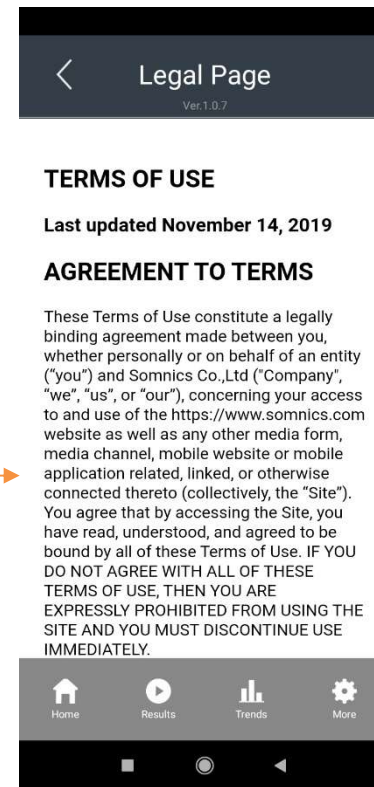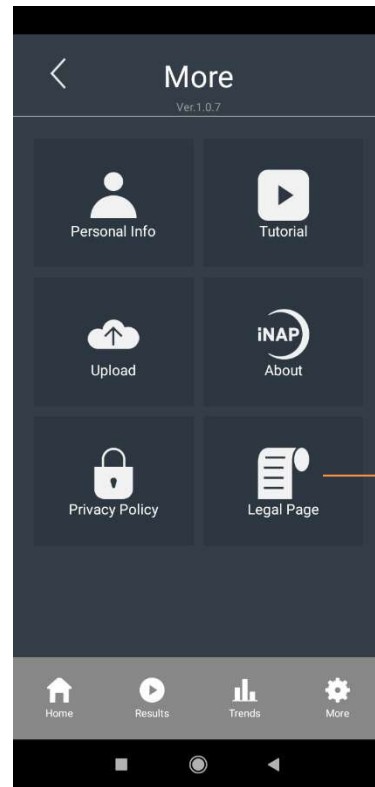

Logout

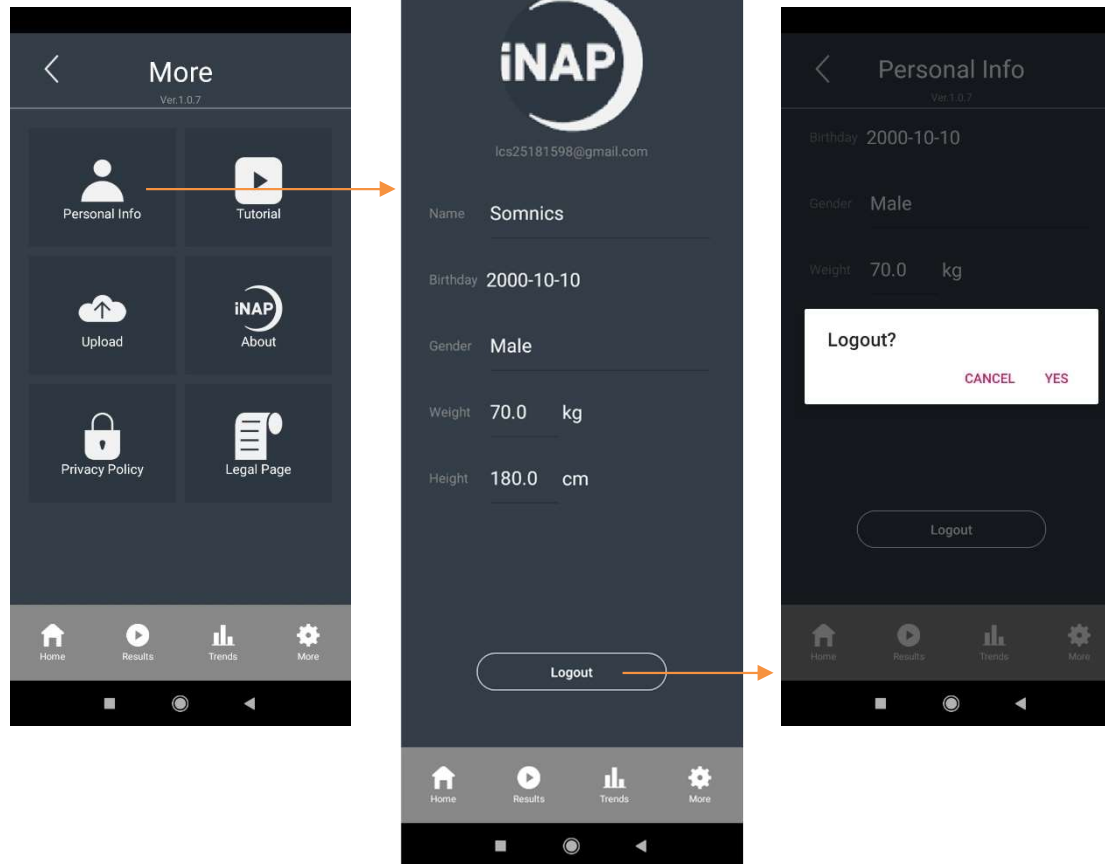

The End

THANK YOU

A: 5F, 22, Sec 2, Sheng-Yi Rd,
Zhubei, Hsinchu, ROC
T: +886-3-5509623
F: +886-3-5503633
W: www.somnics.com

f Facebook.com/somnics
@somnics